

Service Advisory

Solara 3G High-Arm Option Limits Seat Cushion Width

When the Solara 3G High-Arm option is selected, the seat pan width / seat cushion width needs to be reduced by approximately ½ inch from the specified chair width.

Problem Detail: When the High-Arm option is selected on the Solara 3G wheelchair, the front arm mounting cups will interfere with a Pindot ContourU seat pan, or any other seat that has the full specified chair width.



Solution: The Standard-Arm and High-Arm options are provided using the same mounting hardware. For the High-Arm option, the brackets are mounted with the cups above the chair seat rail. For the Standard-Arm option, the hardware is mounted with the cups below the seat rail.


If the High-Arm option causes the front mounting hardware to interfere with seat pan / seat cushion mounting, the chair will need to be changed to the Standard-Arm configuration. This will reduce the maximum arm rest height by 2 ¼ inches.

Changing the Configuration: To change from the High-Arm configuration to the standard configuration:

- Remove the two front arm cup brackets.
- Install the bracket taken from the right side on the left side of the chair, with the cup oriented below the seat rail.
- Install the bracket taken from the left side on the right side of the chair.
- Repeat the procedure of trading left and right brackets on the two rear arm brackets, mounting the cups below the seat rails.

Arm cup
below seat rail.

Arm cup
above seat rail.

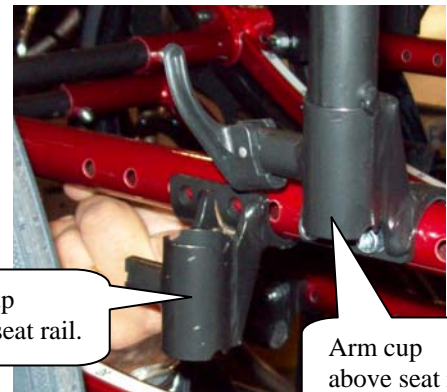

Gary L. Cotton
Pindot Mfg Mgr.



Pindot Customer Service & Technical Support
(800) 451-3553
ElyriaCustomSeating@Invacare.com
899 Cleveland St, Elyria, OH 44036

